## IMPORTANT

**Read carefully and follow the instructions**

FABRIC: 100% COTTON  
HOT/COLD PACK

### INSTRUCTIONS FOR COLD USE:

Place in a plastic bag and into freezer for at least 4 hours before use.

### INSTRUCTIONS FOR HOT USE:

#### TO BE HEATED BY ADULTS ONLY

**Always use a microwave oven that is clean and free of food particles.**

- 1 Fold the pack in two and place it **in the center** of the microwave's turntable.
- 2 Heating time may need to be adjusted, due to differences in power of individual microwaves. To prevent overheating of the pack, microwave in 30-second increments until desired temperature is reached. After each 30 seconds, remove the pack from the microwave and check the temperature for at least 5 seconds. **Total heating should not exceed 2 minutes (maximum of four (4) 30-seconds intervals).** Do not leave pack unattended while heating. Make sure temperature is comfortable to the touch before applying to the skin.

#### COLD

#### HOT

## WARNINGS THIS IS NOT A TOY

**Use only as directed.** This product is not for children under 3 years of age; keep out of their reach. Respect the specified heating time. The pack is made of natural fibers and grains which can catch fire or inflict burns if overheated. Do not leave the product unattended while heating. Allow the pack to cool for at least 1 hour before reheating. Do not heat more than 3 times a day. For more than 3 daily uses, please alternate with a second pack. Heat or cool from room temperature only. Do not directly heat when cold or freeze when hot. Stained or overheated packs must be safely discarded. Always test the temperature of the product before handling. Do not use the pack with therapeutic creams or ointments. Never leave a hot pack unattended on upholstered furniture, under blankets or other flammable materials. Do not use this product if you have any degree of insensitivity to pain or temperature. People with health problems should consult their doctor before use. Stop using the pack if you have an allergic reaction. Never apply this product to an infant, nor to someone who is asleep, unconscious, disabled or otherwise helpless. Do not sleep with the product. This product may lose effectiveness with use, so **it is recommended to change your Magic Bag after a hundred uses.** **Keep the pack away from sources of heat or constant high humidity. This pack is not washable. Keep dry at all times. Store in cool and dry area.**

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Not to be removed until delivered to the consumer

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