
REUSABLE PROTECTIVE FACE MASK INSTRUCTIONS AND CARE

FILTER USE

1. Wash and dry hands before and after touching your mask.
2. Insert a new disposable filter. The deeper bond points should face outward as shown.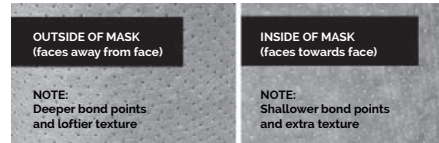

3. Fit and flatten filter so that it follows the inner shape of the mask.
4. Remove and dispose of filter at the end of the day.

ECO GO+ FEATURES 2 DIFFERENT KINDS OF ATTACHMENTS

Model 1

Lay the two elastics above the mask. Place the mask against the face and pull the elastics over your head. The lower elastic should rest at the nape of your neck, while the upper elastic should be on the crown of your head.

Model 2

Place the mask against the face and pull ear loops around ears. Use the toggle to adjust the fit.

ADJUSTING THE FIT OF YOUR MASK

Adjust the elastics as needed to ensure firm fit to your face covering your nose and mouth area. Press the nose clip on both sides until fitted on nose.

Check that mask is fitted to face firmly all around to ensure air flow passes through mask and not in from edges.

To remove the mask, touch only the elastics and pull them over your head. Remove and recycle filter before washing mask following the care instructions indicated below.

FILTRATION

Your mask is equipped with a 2-ply cotton construction as well as a disposable thermoplastic filter.

Used WITHOUT the filter, the filtration level will reach up to 80% for PM0.3 microns.

Used WITH the filter, the filtration level will reach up to 80% for PM0.3 microns; thus it is highly recommended to purchase and use with the Eco Go+ replacement filters available in-store or online at masdelinc.com.

TBC

FILTRATION -CONT'D

If your mask features a removable and disposable filter, remove and dispose of filter in the recycling bin after every use **before** washing your mask.

CARE

- Machine wash with regular detergent in warm water on delicate cycle. Hang to dry.
- Do not use fabric softener.
- Do not put in the dryer.
- Do not iron.
- Do not dry clean.

Note: Due to the natural cotton fibers of your mask, the knitting can become looser after many washes. Industry standards state that protective garments should be replaced after 20-50 washes.

WARNING

A misused mask can become a breeding ground for germs and can cause more problems than it solves. Follow the instructions carefully.

Using a protective face mask can slightly affect your breathing. Do not use the mask when doing physical activity nor when driving. If you feel dizzy or light-headed, remove your mask and breathe normally.

Disclaimer: This product is not intended to be used as or in substitute for a medical device.

WARNING - CONT'D

Masdel Inc does not make any representations or warranties, express or otherwise, regarding the performances of this product, or that the product will help prevent infections or the transmission of viruses or diseases.

This product should not be used in any medical or other setting where the infection risk level is elevated, or where there is exposure to bodily fluids and hazardous materials. This product should not be used in the presence of open flames or flammable liquids or gases. All data provided is nominal and provided for information purposes only.

No liability: The customer understands and agrees that Masdel Inc. cannot be held liable for any injury or damage, direct or indirect, of any kind, in connection with the masks, their use or the manner in which they are worn.

Purchase new masks and/or replacement filters at your local pharmacy or on our website:
masdelinc.com

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**©Masdel Inc. 9081 Rue du Parcours,